



Become the first choice  
in respiratory care.



## N6 Nasal Mask

To remold the image of nasal mask  
family cares



BMC Medical Co., Ltd.

Room 110, Tower A Fengyu Building, No.115 Fucheng Road, Haidian, 100036 Beijing, PEOPLE'S REPUBLIC OF CHINA

Tel: +86-10-5166 3880 / Fax: +86-10-5166 3880 ext. 810 / [en.bmc-medical.com](http://en.bmc-medical.com)

ST-SC-02-21-12-003-V1.0



# BMC N6 Nasal Mask

## Main features



### Under-the-nose Cushion

The innovative under-the-nose cushion minimizes the contact with your nose so that no pressure needs to be exerted on the nose bridge.

### Circular Dispersion Vent

The circular dispersion vent keep the noise as low as 20dB (A) and meanwhile protect your bed partner from being disturbed.



### Modular Main Frame

The main frame is compatible with all cushion sizes.




### Three-point-adjustable Headgear

The adjustable headgear is fit for most of the head sizes. Wider range, more inclusiveness.



## Technical Specifications

Pressure-Flow Curve

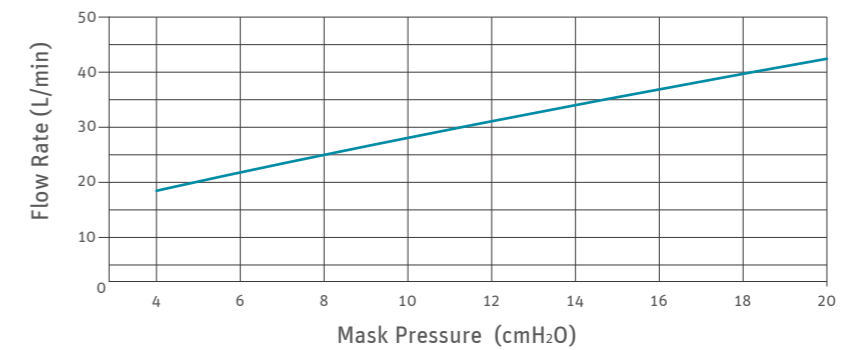

Pressure (cmH <sub>2</sub> O)	4	8	10	12	16	18	20
Flow Rate (L/min)	17	25	28	31	37	40	42

Therapy Pressure 4 to 20 cmH<sub>2</sub>O

Resistance  
Drop in Pressure measured (nominal)  
at 50 L/min: 0.4 cmH<sub>2</sub>O  
at 100 L/min: 1.8 cmH<sub>2</sub>O

Sound  
DECLARED DUAL-NUMBER NOISE EMISSION VALUES in accordance with ISO 4871. The A-weighted sound power level of the mask is 28 dBA, with uncertainty 3 dBA. The A-weighted sound pressure level of the mask at a distance of 1 m is 20 dBA, with uncertainty 3 dBA.